

Fitting the Handle for the Aquarian Bath and Shower Boards

The handle for the Aquarian Bath and Shower Boards may offer enhanced mobility and security to the user during transfers and while washing.

1. Loosen the wingnut that secures the cleat and rotate it towards the middle of the board.



2. Loosen the blue screw and slide the bottom part of the handle assembly over the middle two slats of the board.



3. Position the handle next to the grey bracket and tighten the screw with a coin / key / screwdriver to secure in place.



4. Rotate the cleat back towards the outer edge of the board and proceed with fitting the board as instructed.

