



LOCOMOTOR QUILTED UNI-SLIDE BED

LOCO-191

For more information contact the company you purchased the product from or contact customer services at:

Select Healthcare (UK) Ltd
Unit 2
Albert Road
Rushden
Northants
NN10 0BU

Tel: 01933 411 729
Fax: 01933 410 213

INSTRUCTIONS FOR USE

Place the UNI-SLIDE on the bed in the desired position, ensuring the arrow is pointing towards the head of the bed.

Hoist the patient into bed or use a slide sheet to position the patient on the bed.

If the patient needs re-positioning up the bed, take hold of the top layer of the uni-slide, and using an approved technique slide the patient up the bed on the bed sheet.

Once in position, the construction of the uni-slide will prevent the patient sliding down the bed.

DIMENSIONS:

80cm x 70cm

MATERIALS:

Quilted polyester cotton.
Nylon.

CARE & USE:

Machine wash up to 71°C.
Wash coloureds separately.

IMPORTANT:

All patient moving and handling tasks should be carried out in accordance with approved patient handling techniques and after a full assessment of the patient, the task and the environment in which the operation is to take place.