



**SmitCare**

14 Camberley Business Centre, Bracebridge  
Camberley Surrey GU15 3DP  
Tel: +44 (0)1276 677888 Fax: +44 (0)1276 691565  
Email:sales@smitcare.co.uk




**Instructions for the safe fitting and use of  
SmitCare Metal Grab Rails**

**Code MGR1      18" O/A (450mm)**

**Code MGR2      24" O/A (610mm)**

**Code MGR3      28" O/A (710mm)**

Manufactured in China for SmitCare Ltd

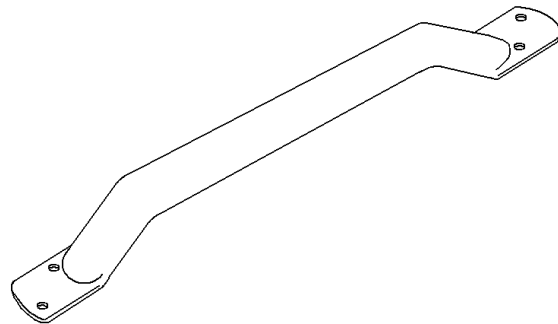

SmitCare Powder Coated Steel Grab Rails may be fitted inside or outside of the house.

**Important Note:**

We recommend that grab rails should be fitted by a competent, qualified tradesman or an Occupational Therapy Technician. We regret we are unable to suggest exactly which fixings should be used as this is determined by the type of wall (brick or stud) or wood that the rail is being screwed onto.

SmitCare are unable to give a weight bearing load figure on Grab Rails because of the unseen dynamic forces which may be applied by the user. In general, it is safe to say that the SmitCare Metal Grab Rails are suitable for users up to 30 stones (190 kg) when used in a straightforward standing or support exercise and secure to a solid wall.

**Cleaning :** Wipe with a damp sponge, using a solution of warm water and household disinfectant. Dry thoroughly. Do not clean with harsh abrasives.

Should you have any questions or comments about this product,  
please contact our Customer Services department  
on: **01276 677888**