
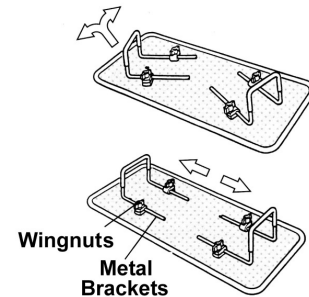


## **Brighton Bathboard** **Fitting and usage instructions**

Product specification:  
Weight Limit: 190kg (30st)  
Board Size: 690 x 310mm  
Bracket adjustment 400 - 600mm  
Depth of bracket 90mm  
Code SNE346

Manufactured in Taiwan for SmitCare

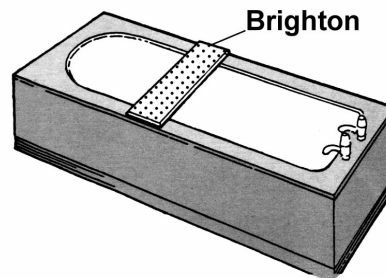

### **Overview**

The Brighton is a strong and secure bathboard manufactured from plastic coated steel, which is easy to fit to a variety of baths, and offers reassurance to the user.

It is important to consider the suitability of this product for the user and to ensure that any potential risks are considered.

### **Fitting the board to the bath**

- 1) Check that the Brighton bathboard is the correct width for your bath. The bathboard must not hang over the edge of the bath, to the extent that any downward pressure on the overhang would cause the bathboard to be dislodged. Also, at least 30mm of the bathboard must rest on the rim of the bath on each side.
- 2) Place the bathboard across the bath at your preferred position with the brackets pointing downwards. Push the brackets out until they are secure against the sides of the bath.
- 3) Tighten the butterfly nuts, finger tight, so they do not move.
- 4) Remove the board and carefully loosen the nuts at one end. Widen the brackets by approximately 2mm and re-tighten. Repeat this procedure at the other end. This will guarantee a tight and safe fit.
- 5) Replace the board on the bath the correct way up. Check that the board is secure and does not slip along the length of the bath. Make sure the brackets are firmly against the walls of the bath, but not so tight that they might cause damage to the bath.

**Note: You should check to make sure the Bathboard is secure in the bath each time you use it.**

Cleaning : Bathrooms can be a source of infections, therefore clean regularly with a solution of mild detergent and household disinfectant in warm water, applied with a damp sponge.  
Dry thoroughly. Do not clean with harsh abrasives.

Questions / comments?

Please contact our customer services department on 01276 677888