



SmitCare

14 Camberley Business Centre
Camberley Surrey GU15 3DP
Tel: +44 (0)1276 677888
sales@smitcare.co.uk
www.smitcare.co.uk

Instructions for the safe use of the Yateley Shower Bench

Weight Limit – 160kg (25st)

Adjustable Height: 490-635mm (20-25½")

Seat Size: 370x280mm (14½ x11")

Width overall: 520mm (20½")

**Footprint Size: 540x390mm min
580x420mm max**

Code: SW127A

Manufactured in China for SmitCare



Important Notes

A risk assessment must be carried out as to the suitability of this product.

The following risks should be considered:

Has the product been correctly assembled?

Will user's health deterioration require further regular reviews?

The Yateley Shower Bench is made from white epoxy coated steel. The aluminium legs are adjustable in height between 490-635mm (20-25½") and are fitted with slip-resistant ferrules. The plastic seat is perforated for drainage and is easily removable for cleaning.

To adjust the height of your Yateley shower bench:

- 1) Lay the frame on the floor or a flat surface and remove the plastic locking **€**-clips from the legs.
- 2) Adjust the legs to the correct height, making sure that all four legs are the same height. Replace the **€**-clips so that the metal point of the clip goes through the leg and can be seen on the other side. NB The metal point of the clip should be on the inside of the frame. Ensure the clip is fully inserted before use.

Cleaning : Wipe down with a soapy solution of warm water, mild detergent and household disinfectant applied with a sponge or soft cloth. Do not use abrasives. Dry thoroughly after cleaning.

Questions / comments?

Please contact our Customer Services department on: 01276 677888