



**Instructions for the safe use and fitting of
SmitCare Threshold Ramps**

Suitable for Wheelchairs and Scooters

Code: TR2 (2 foot Threshold Ramp)

TR3 (3 foot Threshold Ramp)

Manufactured in China for SmitCare

Fig. 1



Important Notes

A risk assessment should be carried out as to the suitability of this product particularly with new users. The following risks should be considered: Entrapment of fingers. Falling/Tripping. Has the product been prepared for safe usage? Will the user be safe on the ramps with their equipment?

Dimensions

Threshold Ramps are available in the following lengths.

Length	Width	Weight	Safe Weight Bearing
2 ft	29 ³ / ₄ " 755 mm	4.8 kg	350 kg
3 ft	29 ³ / ₄ " 755 mm	7.10 kg	350 kg

User / Fitting Instructions

These ramps should not be used for gradients of more than 1 in 12 for a permanent ramping solution (Fig.1).

Benefits

Suitable for most exterior ramping requirements.

Anti-ice draining slots.

Important Notes

These ramps are supplied with pins for a permanent or semi-permanent fixture if required. Should fitting be necessary make sure that they are fitted by a qualified handyman/tradesman or O.T. Technician.

Cleaning

Keep your threshold ramps in good order. Clean regularly with a soapy solution and dry with a cloth.

Should you have any queries please contact our customer services department on: **01276 677888**